



BLADDERSCAN
STANDARD WORK

- 1. GATHER MACHINE Make sure ultrasound gel is with machine
- 2. EXPLAIN Purpose. Non-invasive. Cold gel can be uncomfortable.
- 3. HAND HYGIENE Sanitizer then non-sterile gloves

- 4. PREP PATIENT Supine position
Remove pants and underwear to about 2 inches BELOW symphysis pubis

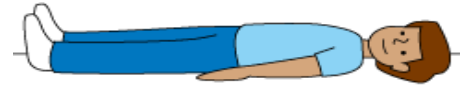

- 5. PREP MACHINE Prior hysterectomy? Change setting to MALE
Align the probe

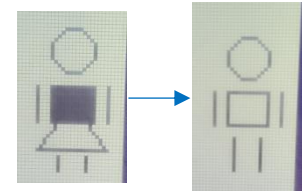

- 6. EXPLAIN Cold gel. Starting procedure.
- 7. PROCEDURE Apply a generous amount of gel to probe.
Place aligned probe 1-2 inches ABOVE the symphysis pubis.
Search for urine in bladder field on machine.
Scan.

- 8. TROUBLESHOOT



- 9. FINISH Capture scan number.
Tell patient amount of urine found.
Wipe off machine with Sani-wipe.
- 10. HAND HYGIENE Remove gloves. Sanitizer.
- 11. DOCUMENT In EMR, bladderscan flowsheet.